
PHILIPS

RESPIRONICS

Sami the Seal

Nebulizer compressor system

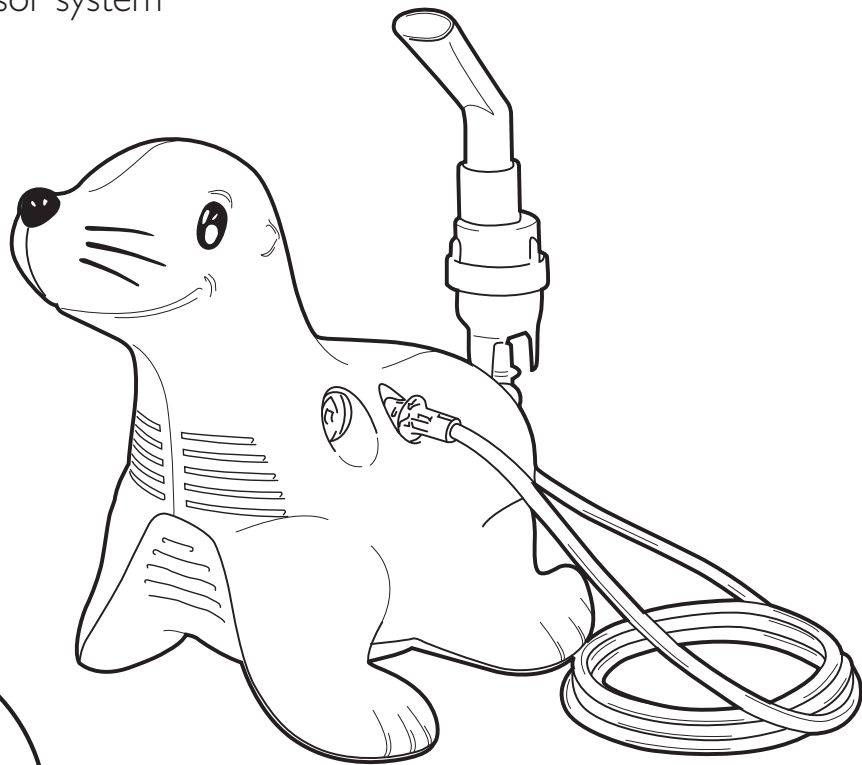
REF 1093268

Sami the Seal

Nebulizer compressor system

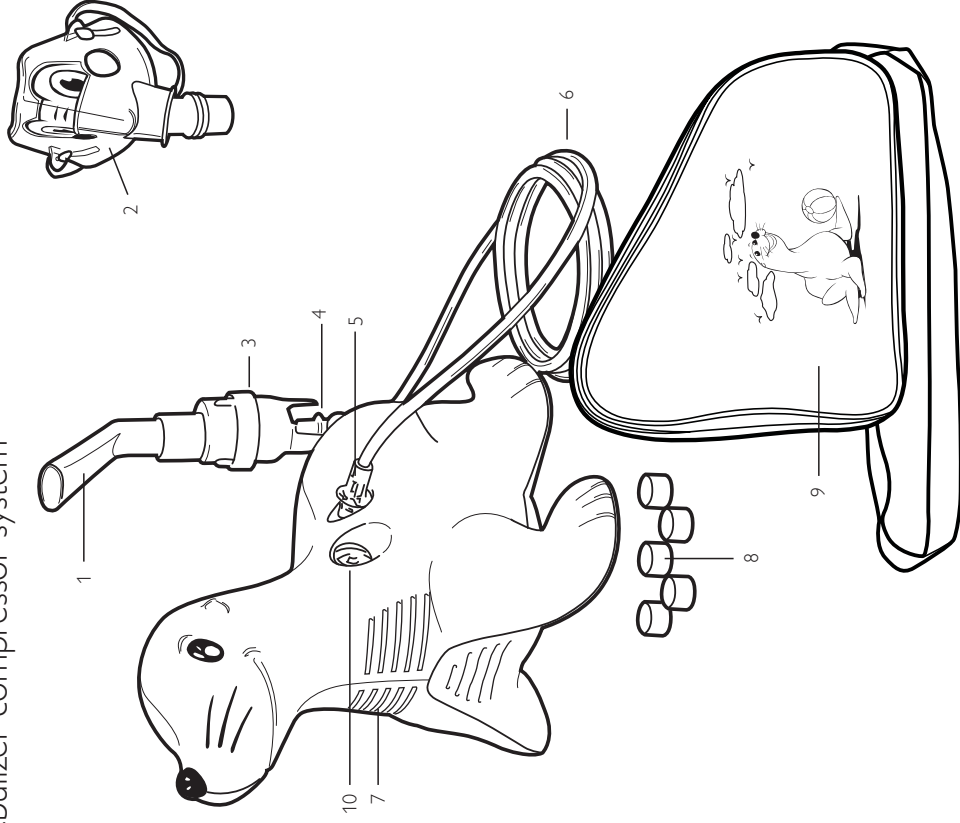Index

English	4
Български	8
Česky	13
Hrvatski	17
Dansk	21
Deutsch	25
Español	30
Eesti	35
Français	39
Ελληνικά	44
Indonesia Bahasa	49
Italiano	54
Latviski	59
Lietuviškai	64
Magyar	68
Nederlands	73
Norsk	78
Polski	83
Português	88
Русский	93
Română	98
Srpski	103
Slovenčina	108
Slovenščina	113
Svenska	117
ภาษาไทย	121
Türkçe	125
Українська	129
العربية	134
中文	139

ENGLISH

1. Nebulizer mouthpiece
2. Tucker the Turtle pediatric mask
3. Medication nebulizer cup
4. Nebulizer holder
5. Compressor air outlet
6. Nebulizer tubing
7. Compressor air openings
8. Air filters
9. Carrying case
10. Air inlet for filter

Intended use

Sami the Seal is an AC-powered air nebulizer compressor system intended to provide a source of compressed air for medical purposes. It is to be used with a pneumatic nebulizer to produce aerosol particles of medication for respiratory therapy for both children and adults.

Unpacking your Sami the Seal system

When you unpack your Sami the Seal from its shipping case:

- Carefully examine the contents.
 - Save the shipping carton.
 - Make sure you have all the content items listed and that they are not damaged.
 - Report anything missing or damaged to Philips Respironics.
- Do not install, maintain or operate this equipment without first reading and understanding these instructions for use. If you are unable to understand any portion of these instructions, contact your healthcare provider or call Philips Respironics Customer Service +1 724 387 4000.

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, safety precautions should always be followed, including the following:

Read all instructions before using

WARNINGS

To reduce the risk of electrocution:

- If the plug does not fit fully into the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. Do not attempt to modify the plug in any way.
- Always unplug this product immediately after using.
- Do not use the unit while bathing.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not place in or drop into water or other liquid.
- Do not reach for product that has fallen into water: Unplug immediately.

To reduce the risk of burns, electrocution, fire or injury to persons:

- A product should never be left unattended when plugged in.
- Close supervision is necessary when this product is used by, on, or near children or physically challenged individuals.
- Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it is has been dropped or damaged, or dropped into water. Return the product to a service center for examination.
- Keep the cord away from heated surfaces.
- Never block the air opening of this product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.
- Never use while sleeping or drowsy.
- Note: It is normal for the device to be warm to the touch when in use.
- If the device has been outside of the operating conditions, allow to come back to normal operating specifications before use.
- Never drop or insert any object into any openings.
- Do not use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered in a closed area such as an oxygen tent.
- No modification of this equipment is allowed.
- Electromagnetic information: Portable and RF communication devices such as cellular phones, pagers, etc. can interrupt operation of electrical medical equipment. For this reason, your compressor must be placed far enough away from these devices to prevent interference. This device meets IEC60601-1-2 standard for electromagnetic compatibility (EMC). EMC data sheets are available from Philips Respironics customer service at +1 724 387 4000 (outside the US).
- Do not stack the compressor with other equipment.

To reduce the risk of electric shock, do not disassemble.

- Do not use compressor and nebulizer separately.

Device safety system

Thermal protector

The device contains a thermal protector which will turn the device OFF if it overheats. **Should the unit overheat, and the thermal protector activates, switch the unit to the off "O" position, then allow the device to cool down for at least 40 minutes before switching the unit back on.**

ENGLISH

How to use your Sami the Seal

- 1 Check the medication nebulizer cup for damage. The medication nebulizer cup enclosed with your Sami the Seal is reusable and should be replaced regularly according to its instructions for use.
- 2 Place your Sami the Seal unit on a firm, flat surface near an electrical outlet and in an area where you can be seated comfortably during your treatment.
- 3 Plug the power cord into the electrical outlet.
- 4 Do not operate the unit without the filter or with a clogged (dirty) or wet filter. Operating the unit with a dirty filter can impair proper operation and cause damage that is not covered by the warranty.
- 5 Wash your hands before preparing your medication.
- 6 Assemble your medication nebulizer cup and add your prescribed medicine following your prescription and the separate nebulizer instructions.
- 7 Connect one end of the tubing to the bottom of the cup and the other end of the tubing to the air outlet on the unit.
- 8 Turn the power switch on "I" to begin your treatment.
- 9 When your treatment is complete, turn the power switch off "O" and unplug the unit.
- 10 Clean the medication nebulizer cup according to its instructions.

Care and cleaning of your Sami the Seal

- 1 Unplug the unit before cleaning.
- 2 Use a damp cloth to wipe down the outside of the compressor at least once each month.
- 3 Refer to nebulizer and accessory instructions for use for cleaning of the accessories and nebulizer.
- 4 Replace the filter when dirty or wet. Replacement filters are available where you purchased this product (Package of 5; Part No. 1083128).
- 5 To change the filter:
 - Twist the filter and remove it by pulling out.
 - The filter should be replaced when it becomes discolored. Do not wash or reuse the filter.
 - When replacing the filter, make sure it is fully inserted.
 - Do not use the compressor without the filter.
- 6 After each cleaning, check that the unit is working properly by verifying that air still comes out of the nozzle when the unit is turned to the "I" position.

The compressor requires no lubrication or maintenance.
 Unauthorized repair voids warranty. There is no service option.
DOUBLE INSULATED

Standard specifications

Electrical requirements	230VAC /50Hz, 0.7 Amp (1093268)
Thermal overload protector	Resettable thermal fuse; functioning temperature 130 °C
Performance	Average flow: 6 lpm @ 69 kPa (0.69 bar) Maximum pressure: 200-300 kPa (2-3 bar)
Replacement intake filter	Part no. 1083128
Size	267 mm L x 155 mm W x 226 mm H
Weight	1.58 kg
Regulatory listing	Safety: IEC 60601-1 EMC: IEC 60601-1-2
Operating mode	Continuous
Operating temperature	↓ 45°F – 104°F (5°C – 40°C)
Operating humidity	RH 10% RH to 95% RH, non-condensing
Operating atmospheric pressure	Performance may vary depending on altitude above sea level, barometric pressure and temperature
Storage/transport temperature	↓ -13°F – 158°F (-25°C – 70°C)
Storage/transport humidity	RH 10% RH to 95% RH, non-condensing
Storage/transport atmospheric pressure	P 690-1060 hPa

Symbol glossary

- | | | | | | |
|--|-----------------------------|--|---------------------|--|----------------------------|
| | ON (power) | | ATTENTION | | Class II, double insulated |
| | OFF (power) | | Alternating current | | Type BF applied parts |
| | Serial number | | Separate collection | | Indoor use only |
| | Follow instructions for use | | Manufacturer | | Temperature limitation |
| | Atmospheric pressure | | Humidity | | |

IP21 Housing Ingress Protection rate: IP21 (protected from the penetration of solid bodies with dimensions greater than 12 mm. Protected from the penetration of vertically falling water drops)

Warranty

Respironics, Inc warrants the Sami the Seal compressor to be free from defects in materials and workmanship under normal use and operation for a period of 3 years from date of purchase from Respironics, Inc. The warranty is limited to replacement, at Respironics, Inc's sole option, of any such component or equipment claimed to be defective when claim is shown to be bona fide by evaluation by Respironics, Inc. This warranty does not extend to any components or equipment subjected to misuse, improper operation, accidental damage, or unauthorized repairs, and is not extended to charges of or for labor repairs. All items returned must be properly packaged and shipped, prepaid, by the distributor servicing the unit. Respironics, Inc shall not be liable to purchaser or others for loss of use of equipment or for indirect, or incidental or consequential damages that might arise.

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